

kogan

www.kogan.com



Operating Instructions

To test your blood alcohol content:

1. Open the rear battery cover and insert 2 AAA 1.5V batteries.
2. Press the Reset Button as shown above to wake up and reset the detector. When ready, the green Ready light will turn on.
3. Blow into the intake on the top left of the unit.
4. Your calculated BAC will be shown by the following:
 - a. If the alcohol concentration in your blood is ≥ 0.02 and $< 0.05\%$, the yellow LED will turn on.
 - b. If the alcohol concentration in your blood is ≥ 0.05 , the yellow LED and the red LED will turn on.

To use the built-in LED torch, press the LED Torch Button as shown above.

Please note, the testing accuracy may be influenced by your environment (e.g. a crowded bar).

Always remember that each person has their own tolerance for alcohol, and this device should be used for reference only. Do not use this device to determine if you are legally able to drive.